



User manual

Seat cushion
- Standard/Individual

Introduction

Our seat cushion - Standard/Individual covers the range between a standard seat cushion and a completely individually manufactured Anatomic seat cushion after scanning.

The cushion is made of a pressure-relieving foam, which makes it easy to modify and adapt it to the patient's needs and conditions.

Anatomic Seat Cushion Standard is a product designed to be individually adapted to the patient by the Aid Center.

If you need additional information, you are welcome to contact your local supplier or Anatomic SITT AB.

Safety precautions

In order to use the seat cushion correctly, it is important that you read this manual. This concerns to all persons assisting to the patient.

When replacing a seat cushion, it may be necessary to consult with a therapist or other medically qualified person to make minor adjustments to your wheelchair.

The skin should be checked regularly to detect any redness, especially around the ischial tuberosity. Contact your doctor immediately if persistent redness is noted to avoid pressure sores.

Do not use the seat cushion if the foam on the inside of the cover is wrinkled or folded. This can create a pressure point that can cause a pressure sores. Be careful with the cover so that you do not wrinkle or fold the foam during transport.

Never expose your seat cushion for any sharp objects.

Never store your seat cushion in extreme heat or direct sunlight for an extended period of time.

**Note!**

The seat cushion is tailor made for the user. Therefore, be aware of any redness and pressure marks, etc. If problem appears, contact the responsible prescriber.



Note! If you are not satisfied with the seat cushion or theres any problem, please contact your prescriber, local provider or Anatomic SITT AB. Regular follow ups is necessary.

Placement of the cushion

The seat cushion should lie firmly against the surface/basis without slipping.
It is suggested that the seat cushion is fastened with strong velcro straps to the base.

Care- and washing instructions

The cover can be machine washed at 60 °C according to the following washing instructions. Let it dry at room temperature, or in a low-temperature dryer. Do not place directly against a heat source. Do not dry in a dryer.



The foam can be washed by rinsing plenty of water through the foam. Squeeze out the water and allow the foam to air dry or dry in a drying cabinet on low heat until it is completely dry. ka eller torka i torkskåp på låg värme tills det är helt torrt.

Recycling

Disassemble a used part or product. The parts must be separated by material type and left at the local recycling station.



Note!

A processed seat cushion is not a CE-marked product and should be treated as a specially made product.



The product is manufactured by Anatomic SITT AB

Address Anatomic SITT AB, Box 6137, 600 06 Norrköping, SE

Visiting Anatomic SITT AB, Terminalgatan 1, 603 61 Norrköping, SE

Phone 011-161800 **Fax** 011-162005

E-mail info@anatomicsitt.com **Web** anatomicsitt.com